





BE-4301

Installation Guide

1 Fixed to wall

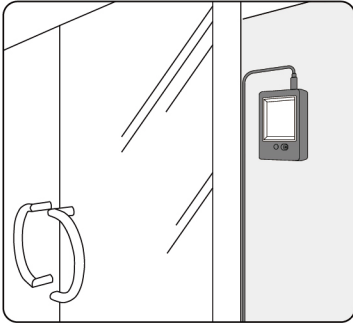

2 Mounted on tripod

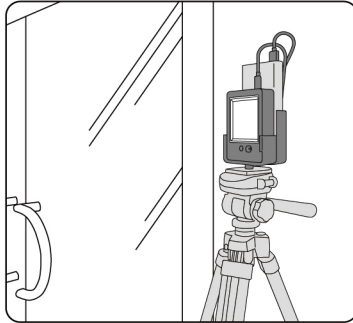

* Tripod and mobile power not included

Instructions

When the palm of the hand or face is close to the sensor area, stay within the measurement range for 2 seconds and the measurement result will be announced automatically in English.


Note

- 1 Do not leave it in a high temperature or high humidity environment for children to play with.
- 2 Do not leave it in a sunny or rainy place to avoid damage to the product.
- 3 Do not drop the product by gravity, or hit it as this may cause damage.
- 4 Pay attention to the daily maintenance of the product and clean the dust of the sensor in order to obtain the best measurement accuracy.
- 5 After power on, the unit switch button and memory search button cannot be pressed at the same time, which will cause the data of the product factory calibration to be erased and cannot be used!




BE-4301

Description

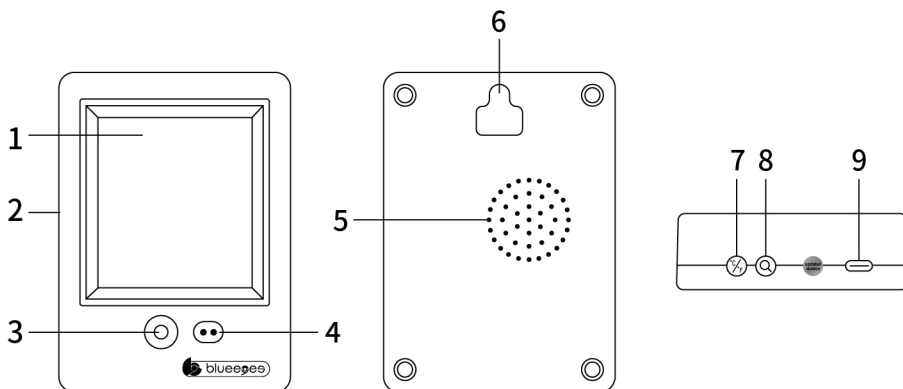

- ① Display Screen
- ④ Light sensor
- ⑦ Unit switch button
- ② Mainframe
- ⑤ Speaker
- ⑧ Memory search button
- ③ Temperature sensor
- ⑥ Wall mount hole
- ⑨ Power input port

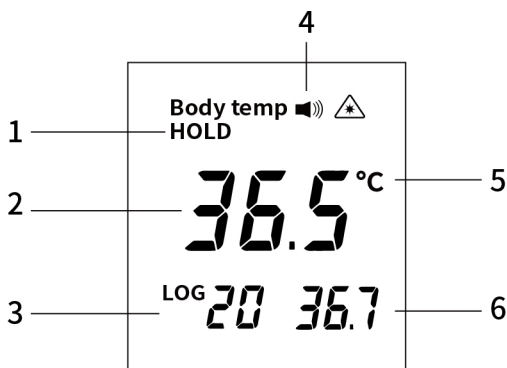

- ① Hold
- ② Temperature
- ③ Memory group
- ④ Voice report
- ⑤ Temperature Unit
- ⑥ Memory Temperature